

Sample Collection and Viral Transport Solution



For more details, visit: <http://info.abmgood.com/Viral-Transport>

abm's Sample Collection and Viral Transport Solution safely preserves and transports infectious biological specimens without the need for dry ice.

Our solution is specially formulated to inhibit bacterial and fungal growth and immediately stabilizes and inactivates viruses collected from nasopharyngeal or oropharyngeal swabs.

Key features:



Inactivates Viruses




Inhibits Bacterial & Fungal Growth



No Dry Ice Needed

Procedure:

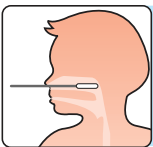


Insert swab into nostril and to the back of the nasopharynx. Withdraw swab in a rotating motion.



Uncap the Viral Transport Solution tube and place swab inside.



Break swab shaft at indicated breaking point.

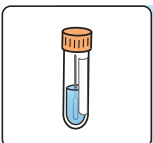

Tightly secure the cap and label with patient information.



KIT COMPONENTS

Viral Transport Solution Flocked Swabs

SAMPLE TYPES

Naso/Oropharyngeal Specimens

Ordering Information

Cat. No.	G631
Kit Size	100 Kits/Package
Price	Inquire